

VACUETTE® TIPGUARD Safety Tube Holder

Greiner Bio-One presents the VACUETTE® TIPGUARD Safety Tube Holder with automatic withdrawal of needle back into the plastic housing

Comfort and safety are the key features of this latest product innovation. After routine blood collection procedure, the safety mechanism is activated whilst the needle is still in the vein. By pressing the two blue release triggers on the top of the holder, the spring mechanism automatically withdraws the needle back into the holder. The needle is then safely enclosed inside the holder, ready for disposal with no risk of injury.

The VACUETTE® TIPGUARD Safety Tube Holder was especially developed for patients at increased risk of infection. To ensure optimal function, the holder should only be used with VACUETTE® blood collection needles.

The advantages:

- Practical, as no change to technique is required
- Simple activation due to safety mechanism
- Automatic needle withdrawal
- Contaminated needle safely enclosed in holder prevents risk of injury

Item n°.	Description	Packaging	
		Inner	Outer
450240	VACUETTE® TIPGUARD Safety Tube Holder	60 pcs.	480 pcs.